


VISSCO HEATING PAD (ACTIVE 4010 ORTHOPAEDIC HEATING BELT)

Electric heating belt with dual thermostat and 3 heat levels—delivers safe, uniform warmth to relieve joint pain, improve circulation, and support faster recovery.

Features

- **WHEN TO USE**
- The heating belt enables uniform warmth to the affected areas resulting in instant joint pain relief. Heat therapy causes vasodilation of blood vessels that helps to improve blood circulation, relieves pain and improves recovery over time.
- **SCIENTIFIC & SAFE DESIGN**
- The dual thermostat helps to control the temperature and the heating level. The electric controller switch with 3 temperature heat levels. The product is designed with 6 layers of insulation that ensures 100% safety.
- **HOW TO USE**
- Place the heating belt around the painful area. Attach the heating belt to a power source & adjust the level of heating as per your comfort. The treatment time can be 15-20 minutes.
- **PRODUCT CARE**
- Do not use while sleeping, Do not wash, Do not iron, Hand wash cover only, Do not fold, Keep away from children. Do not use more than 1 hour.

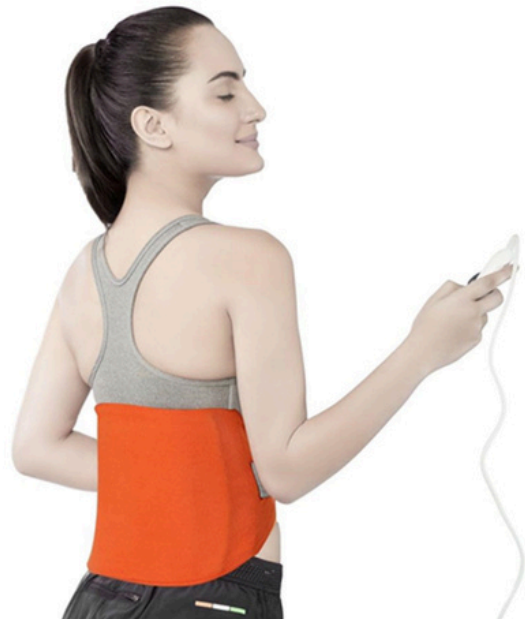

ADDRESS

99, Princess Street, Sancheti
Bhavan, Opp. Earth Baug Bldg,
Mumbai 400 002

CONTACT

onymed33@gmail.com
+91 97690 31545